

Product Specification of:

## Medical Oxygen Sensor

**Type:** M-09

**Part Number:** 410013

**Nuova ID:** E-09/0

RoHS compliant and SVHC free

Meets the applicable requirements of ISO 80601-2-55

Comes with CE marking. further regulatory registrations available upon request

Produced under EN ISO 13485 Quality Management System

### DOCUMENT PURPOSE

The purpose of this document is to present the performance specification and key features of the sensor.

This document should be used in conjunction with the Operating Manual of the instrument and the Product Safety Data Sheet of the sensor.

### KEY FEATURES

Single-cathode galvanic gas sensor that shows high signal stability at low cross sensitivities to anesthesia gases combined with superior linearity over the entire measurement range.



High Signal Stability



Superior Linearity



Wide Measurement Range



Low Cross-Sensitivities

## TECHNICAL SPECIFICATIONS

### MEASUREMENT

Operating Principle:	Partial pressure electrochemical cell	
Measurement Range:	0 ... 100 Vol.%O <sub>2</sub>	
Initial Output Signal:	9.5 ... 14 mV	at dry ambient air
Response Time t <sub>90</sub> :	< 18 s	
Signal Drift (long-term):	≤ ± 1 Vol.%O <sub>2</sub> /month	at ambient air
Linearity Error:	≤ ± 3 %	at 100 Vol.% O <sub>2</sub> applied for 5 min
Zero Signal Offset:	≤ 200µV	at 100 Vol.% N <sub>2</sub> applied for 5 min
Repeatability Error:	≤ ± 1 Vol.% O <sub>2</sub>	at 100 Vol.% O <sub>2</sub> applied for 5 min
Influence of Gas Humidity:	- 0.03 % of signal per %rH	
Signale Temperature Compensation	NTC on sensor PCB	
Cross-Sensitivity:	Fullfills DIN EN ISO 8060-1-2-55	

### ELECTRICAL

Electrical Connector	3-Pin Molex on wire
Recommended Load Resistor:	≥ 10 kOhm

### MECHANICAL

Weight:	≤ 23 g
Material in Contact with Media:	PA, PPS, PTFE, Stainless Steel
Gas Connector:	fits for M16x1 DIN 13 or 5/8-24 UNEF

### ENVIRONMENTAL

Operating Temperature Range:	10 ... 40 °C	
Ambient Pressure Range:	700 ... 1250 hPa	
Ambient Humidity Range:	up to 100 %rH	non-condensing

### LIFETIME

Expected Operating Life:	6 years	at ambient air, depending on application
Nominal Sensor Life:	500 000 Vol.%h O <sub>2</sub>	at ambient air, depending on application

**Important Note:** All characteristics are based on conditions at 25 °C, 50 %rH, 1013 hPa and a gas flow of > 2.5 L<sub>g</sub>/min. For sensor performance data under other conditions, contact ITG.

## STORAGE CONDITIONS IN UNOPENED ORIGINAL PACKAGE

Ambient Temperature Range:	15 ... 25 °C	recommended
	-20 ... 50 °C	maximum ( $\leq 10$ h)
Ambient Pressure Range:	700 ... 1250 hPa	
Ambient Humidity Range:	50 ... 100 %rH	recommended, non-condensing, maximum one week

## RELATED PRODUCTS

Product	Part Number	Other Specifics
O <sub>2</sub> Sensor M-09	41 00 13	—
O <sub>2</sub> Sensor M-09T	41 01 13	translucent housing
O <sub>2</sub> Sensors M-09F	41 02 13	Fluke connector on wires

## Cleaning and Disinfection

The sensor outer housing can be cleaned with a dry wipe. Do not wipe the sensor's gas entrance side. Do not use any chemical disinfectant or sanitizer on the sensor.

## Poisoning

ITG sensors are designed to operate in a wide range of environments. For optimal sensor lifetime and performance it is important that exposure to high concentrations of solvent vapors is avoided during storage, installation into instruments and operation of the sensor. Do not use adhesives directly on or near the sensor as the solvents may cause stress corrosion on the plastic parts.

## Intended Use

The oxygen sensors for use in medical environments are intended to be oxygen-sensing components to determine the oxygen content in breathing gas mixtures. They deliver an electrical output signal that is processed by other finished medical devices, for which they are a limited-life accessory or a spare part.

## Stabilisation Time

When installing a new sensor refer to the instrument manual for stabilization time before calibration.  
If not specified otherwise wait at least 15 minutes to ensure the sensor has stabilized in the instrument.

## Calibration Interval

ITG sensors are designed to have minimal signal drift over their functional lifetime. For optimal performance and maximum measurement accuracy however they should be calibrated before each use.

## If the Sensor is dropped

If a sensor is dropped, please check for visible mechanical damage of the sensor or if the sensor is leaking electrolyte. If this is the case, do not try to install the sensor into the device. Take safety precautions and immediately dispose the cell.

If the sensor shows no visible damage it should be placed in quarantine for at least 24 hours. Afterwards a follow-up check made by a two point calibration should be done.

## Installation in Device

A gas tight sealing is ensured when the sensor is screwed in hand-tight. Don't use any mechanical tools to install the sensor. Using excessive force may damage the sensor.

Optimal mounting position of the sensor is when the gas sensing area faces downwards. A horizontal position is acceptable. It is not recommended to use the sensor with the gas sensing area facing upward.

Connection should be made via recommended electrical and mechanical connectors only. The specified load resistance must be taken into account. Soldering the sensor will damage it and void the warranty. Please contact ITG for further information.

Since temperature has an influence on the output signal it is not advisable to place any heat or vibration sources (i.e. electric- pumps or valves, coolers, etc.) in close proximity to the cell.

Avoid proximity of the cell to any EMC radiating units as those emit frequencies which might interfere with the sensor's electrical connection and connecting cable.

## RFI/EMI Susceptibility

ITG sensors contain metal parts and might be susceptible to RFI or EMI. Before use in MRI environments please contact ITG for further information.

## Disposal

At the end of the sensors lifetime the sensor should not be disposed of in normal public waste as it may contain hazardous materials and caustic electrolyte (for more information refer to the PSDS). Please contact your local authorities for environmental legislation to relevant local waste disposal.

## Description of Pictograms



Manufacturer



Medical device



Model number  
Customer Sensor Type/ Customer article number



Sensor part number of the manufacturer  
ITG sensor type / ITG article number




Serial number




Contains hazardous substances



Indicates the temperature limits to which the medical device can be safely exposed

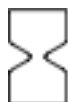

Expiration Date (DD-MM-YYYY)

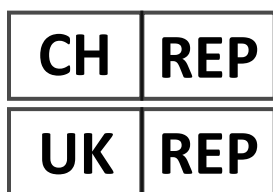

Responsible representative in the territory





Federal law restricts this device to sale by or on the order of a licensed practitioner (U.S. only)



CE Marking with certifying notified body number

## TECHNICAL DRAWING

Sensor	ITG Part number	Color			Electrical pin assignment		
		Cap	Housing	Threaded ring	Pin 1	Pin 2	Pin 3
M-09	410013	-	white	white	minus (-) orange	nc	plus (+) yellow

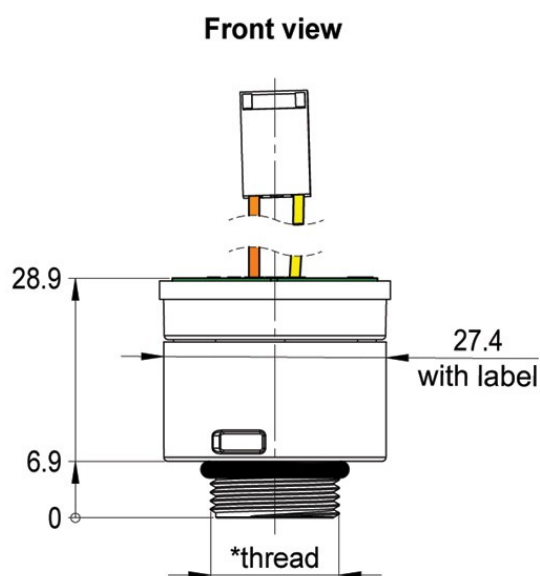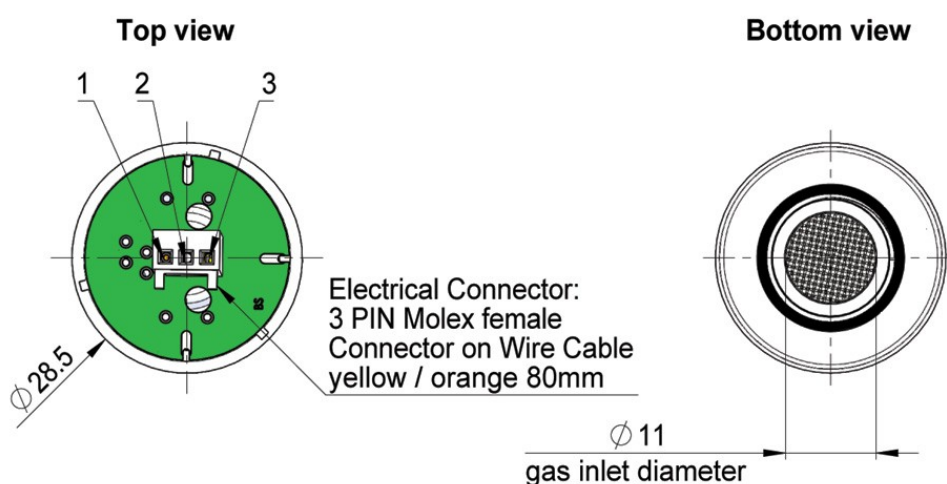

[\* thread is compatible with DIN 13 M16x1 and UNEF 5/8-24]

Dimension unit: mm

Dimension tolerances: linear  $\pm 0.5$  mm , diameter  $\pm 0.3$  mm