



Product Specification of:

# **Medical Oxygen Sensor**

Type: M-12A

Part Number: 410217

Nuova ID: E-17/A2A

RoHS compliant and SVHC free

Meets the applicable requirements of ISO 80601-2-55

Comes with CE marking. further regulatory registrations available upon request

Produced under EN ISO 13485 Quality Management System

## DOCUMENT PURPOSE

The purpose of this document is to present the performance specification and key features of the sensor.

This document should be used in conjunction with the Operating Manual of the instrument and the Product Safety Data Sheet of the sensor.

# KEY FEATURES

Long-life galvanic gas sensor with high signal stability at low cross sensitivity to anesthesia gases combined with superior linearity over the entire measurement range.



This data sheet is subject to change without prior notice Specification\_M-12A\_Rev\_1

Page 1 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany



Product Specification of: Medical Oxygen Sensor Type: M-12A Part Number: 410217 Nuova ID: E-17/A2A



# TECHNICAL SPECIFICATIONS

#### **MEASUREMENT**

Ambient Humidity Range:	up to 100 %rH	non-condensing
Ambient Pressure Range:	700 1250 hPa	
Operating Temperature Range:	10 40 °C	
ENVIRONMENTAL		
Gas Connector:	fits for M16x1 DIN 13 or 5/8-24 UNEF	
Material in Contact with Media:	PA, PPS, PTFE, Stainless Steel, NBR	
Weight:	≤ 24 g	
MECHANICAL		
Recommended Load Resistor:	≥ 10 kOhm	
Electrical Connector	3-Pin Molex	
ELECTRICAL		
Cross-Sensitivity:	fullfills DIN EN ISO 80601-2-55	
Signale Temperature Compensation	NTC on sensor PCB	
Influence of Gas Humidity:	- 0.03 % of signal per %rH	
Zero Signal Offset: Repeatability Error:	$\leq 0.3 \text{ Vol.}\%\text{O}_2$ $\leq \pm 1 \text{ Vol.}\%\text{O}_2$	at 100 Vol.%N <sub>2</sub> applied for 5 min at 100 Vol.%O <sub>2</sub> applied for 5 min
Linearity Error:	≤±3%	at 100 Vol.%O <sub>2</sub> applied for 5 min
Signal Drift (long-term):	$\leq \pm 1 \text{ Vol.%O}_2/\text{month}$	at ambient air
Response Time t90:	< 15 s	
Initial Output Signal:	8.0 12 mV	at dry ambient air
Measurement Range:	0 100 Vol.%O <sub>2</sub>	
Operating Principle:	Partial pressure electrochemical cell	

Important Note: All characteristics are based on conditions at 25 °C, 50 %rH, 1013 hPa and a gas flow of > 2.5 L<sub>s</sub>/min. For sensor performance data under other conditions, contact ITG.

This data sheet is subject to change without prior notice S

Specification\_M-12A\_Rev\_1

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 :: Fax: +49 (0)3841 220052 2 :: E-Mail: info@itg-wismar.de



Product Specification of: Medical Oxygen Sensor Type: M-12A Part Number: 410217 Nuova ID: E-17/A2A



Ambient Temperature Range:	15 25 °C	recommended
	-20 50 °C	maximum (≤ 10 h)
mbient Pressure Range:	700 1250 hPa	
mbient Humidity Range:	50 100 %rH	recommended non-condensing
	0 30 %rH	maximum one week
ELATED PRODUCTS		
roduct	Part Number	Other Specifics
D <sub>2</sub> Sensor M-12	41 00 17	-
2 Sensor M-12	41 01 17	double pack
D <sub>2</sub> Sensor M-12T	47 00 29	translucent housing

#### **Cleaning and Disinfection**

The sensor outer housing can be cleaned with a dry wipe. Do not wipe the sensor's gas entrance side. Do not use any chemical disinfectant or sanitizer on the sensor.

#### Poisoning

ITG sensors are designed to operate in a wide range of environments. For optimal sensor lifetime and performance it is important that exposure to high concentrations of solvent vapors is avoided during storage, installation into instruments and operation of the sensor. Do not use adhesives directly on or near the sensor as the solvents may cause stress corrosion on the plastic parts.

## Intended Use

The electrochemical oxygen sensors for use in medical technology are used as accessories with a limited shelf life exclusively in conjunction with other medical products (ventilators, incubators, anesthesia machines, portable oxygen monitors, oxygen therapy devices and gas mixing units) so that they can fulfill the manufacturer's intended purpose.

The oxygen sensors measure the oxygen partial pressure in gas mixtures. Commissioning and use is carried out exclusively by expert clinic or service personnel. Direct contact with these items occurs during installation or maintenance by trained personnel.

### Stabilisation Time

When installing a new sensor refer to the instrument manual for stabilization time before calibration.

If not specified otherwise wait at least 15 minutes to ensure the sensor has stabilized in the instrument.

## **Calibration Interval**

ITG sensors are designed to have minimal signal drift over their functional lifetime. For optimal performance and maximum measurement accuracy however they should be calibrated before each use.

This data sheet is subject to change without prior notice

Specification\_M-12A\_Rev\_1

Page 3 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 .: Fax: +49 (0)3841 220052 2 .: E-Mail: info@itg-wismar.de

www.itg-wismar.de



Product Specification of: Medical Oxygen Sensor Type: M-12A Part Number: 410217

Nuova ID: E-17/A2A



## If the Sensor is dropped

If a sensor is dropped, please check for visible mechanical damage of the sensor or if the sensor is leaking electrolyte. If this is the case, do not try to install the sensor into the device. Take safety precautions and immediately dispose the cell.

If the sensor shows no visible damage it should be placed in quarantine for at least 24 hours. Afterwards a follow-up check made by a two point calibration should be done.

## Installation in Device

A gas tight sealing is ensured when the sensor is screwed in hand-tight. Don't use any mechanical tools to install the sensor. Using excessive force may damage the sensor.

Optimal mounting position of the sensor is when the gas sensing area faces downwards. A horizontal position is acceptable. It is not recommended to use the sensor with the gas sensing area facing upward.

Connection should be made via recommended electrical and mechanical connectors only. The specified load resistance must be taken into account. Soldering the sensor will damage it and void the warranty. Please contact ITG for further information.

Since temperature has an influence on the output signal it is not advisable to place any heat or vibration sources (i.e. electric- pumps or valves, coolers, etc.) in close proximity to the cell.

Avoid proximity of the cell to any EMC radiating units as those emit frequencies which might interfere with the sensor's electrical connection and connecting cable.

## **RFI/EMI** Susceptibility

ITG sensors contain metal parts and might be susceptible to RFI or EMI. Before use in MRI environments please contact ITG for further information.

#### Disposal

At the end of the sensors lifetime the sensor should not be disposed of in normal public waste as it may contain hazardous materials and caustic electrolyte (for more information refer to the PSDS). Please contact your local authorities for environmental legislation to relevant local waste disposal.

This data sheet is subject to change without prior notice Specification\_M-12A\_Rev\_1

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany




Product Specification of: Medical Oxygen Sensor Type: M-12A Part Number: 410217 Nuova ID: E-17/A2A



# TECHNICAL DRWAING

	Sensor ITG		Color			Electrical pin assignment		
	5611501	Part number	Сар	Housing	Threaded ring	Pin 1	Pin 2	Pin 3
	M-12A	410217	white	white	white	minus (-)	minus (-)	plus (+)

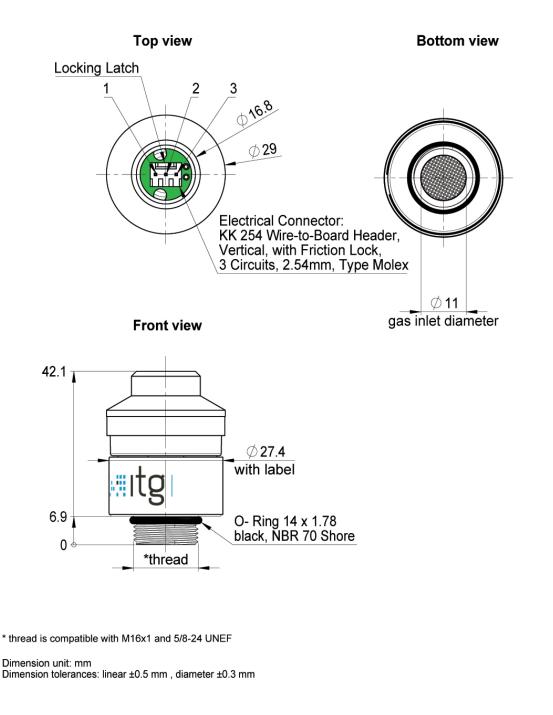

This data sheet is subject to change without prior notice Specification\_M-12A\_Rev\_1

Page 5 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 :: Fax: +49 (0)3841 220052 2 :: E-Mail: info@itg-wismar.de