



Product Specification of:

Medical Oxygen Sensor

Type: M-04C

Part Number: 410206

Nuova ID: E-17/C

RoHS compliant and SVHC free

Meets the applicable requirements of ISO 80601-2-55

Comes with CE marking. further regulatory registrations available upon request

Produced under EN ISO 13485 Quality Management System

DOCUMENT PURPOSE

The purpose of this document is to present the performance specification and key features of the sensor.

This document should be used in conjunction with the Operating Manual of the instrument and the Product Safety Data Sheet of the sensor.

KEY FEATURES

Long-life galvanic gas sensor with high signal stability at low cross sensitivity to anesthesia gases combined with superior linearity over the entire measurement range.









High Signal Stability

Superior Linearity

Wide Measurement Range

Long Lifetime

This data sheet is subject to change without prior notice

Specification_M-04C_Rev_1

Page 1 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 .: Fax: +49 (0)3841 220052 2 .: E-Mail: info@itg-wismar.de www.itg-wismar.de



Product Specification of: Medical Oxygen Sensor

Type: M-04C

Part Number: 410206

Nuova ID: E-17/C



TECHNICAL SPECIFICATIONS

IVI	EA		~	IVI	_	м і	

Operating Principle:	Partial pressure electrochemical cell	
Measurement Range:	0 100 Vol.%O ₂	
Initial Output Signal:	7.5 13.5 mV	at dry ambient air
Response Time t90:	< 12 s	
Signal Drift (long-term):	≤ ± 1 Vol.%O ₂ /month	at ambient air
Linearity Error:	≤ ± 3 %	at 100 Vol.%O ₂ applied for 5 min
Zero Signal Offset:	≤ 0.3 Vol.%O ₂	at 100 Vol.%N ₂ applied for 5 min
Repeatability Error:	≤ ± 1 Vol.%O ₂	at 100 Vol.%O ₂ applied for 5 min
Influence of Gas Humidity:	- 0.03 % of signal per %rH	
Signale Temperature Compensation	NTC on sensor PCB	

ELECTRICAL

Cross-Sensitivity:

Electrical Connector	3-Pin Molex Gold Plated
Recommended Load Resistor:	≥ 10 kOhm

fullfills DIN EN ISO 80601-2-55

MECHANICAL

Weight:	≤ 28 g
Material in Contact with Media:	PA, PPS, PTFE, Stainless Steel NBR
Gas Connector:	fits for M16x1 DIN 13 or 5/8-24 UNEF

ENVIRONMENTAL

Operating Temperature Range:	10 40 °C			
Ambient Pressure Range:	700 1250 hPa			
Ambient Humidity Range:	up to 100 %rH	non-condensing		

LIFETIME

Expected Operating Life:	6 years	at ambient air, depending on application
Nominal Sensor Life:	750 000 Vol.%h O ₂	at ambient air, depending on application

Important Note: All characteristics are based on conditions at 25 °C, 50 %rH, 1013 hPa and a gas flow of > 2.5 Ls/min. For sensor performance data under other conditions, contact ITG.

This data sheet is subject to change without prior notice

Specification_M-04C_Rev_1

Page 2 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 .: Fax: +49 (0)3841 220052 2 .: E-Mail: info@itg-wismar.de www.itg-wismar.de



Product Specification of:

Medical Oxygen Sensor

Times No. 046

Type: M-04C

Part Number: 410206

Nuova ID: E-17/C



STORAGE CONDITIONS IN UNOPENED ORIGINAL PACKAGE

Ambient Temperature Range: $15 \dots 25 \, ^{\circ}\text{C}$ recommended $-20 \dots 50 \, ^{\circ}\text{C}$ maximum ($\leq 10 \, \text{h}$)

Ambient Pressure Range: 700 ... 1250 hPa

Ambient Humidity Range: 50 ... 100 %rH recommended non-condensing

0 ... 30 %rH maximum one week

RELATED PRODUCTS

Product	Part Number	Other Specifics
O ₂ Sensor M-04	41 00 06	-
O ₂ Sensor M-04T	47 00 04	translucent housing
O ₂ Sensor M-04C	41 02 06	Initial Output Signal: 7.5 13.5 mV, flat seal
O ₂ Sensor M-04CT	41 03 06	Initial Output Signal: 7.5 13.5 mV, flat seal, translucent housing

Cleaning and Disinfection

The sensor outer housing can be cleaned with a dry wipe. Do not wipe the sensor's gas entrance side. Do not use any chemical disinfectant or sanitizer on the sensor.

Poisoning

ITG sensors are designed to operate in a wide range of environments. For optimal sensor lifetime and performance it is important that exposure to high concentrations of solvent vapors is avoided during storage, installation into instruments and operation of the sensor. Do not use adhesives directly on or near the sensor as the solvents may cause stress corrosion on the plastic parts.

Intended Use

The electrochemical oxygen sensors for use in medical technology are used as accessories with a limited shelf life exclusively in conjunction with other medical products (ventilators, incubators, anesthesia machines, portable oxygen monitors, oxygen therapy devices and gas mixing units) so that they can fulfill the manufacturer's intended purpose.

The oxygen sensors measure the oxygen partial pressure in gas mixtures.

Commissioning and use is carried out exclusively by expert clinic or service personnel. Direct contact with these items occurs during installation or maintenance by trained personnel.

Stabilisation Time

When installing a new sensor refer to the instrument manual for stabilization time before calibration.

If not specified otherwise wait at least 15 minutes to ensure the sensor has stabilized in the instrument.

Calibration Interval

ITG sensors are designed to have minimal signal drift over their functional lifetime. For optimal performance and maximum measurement accuracy however they should be calibrated before each use.

This data sheet is subject to change without prior notice

Specification_M-04C_Rev_1

Page 3 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 .: Fax: +49 (0)3841 220052 2 .: E-Mail: info@itg-wismar.de www.itg-wismar.de



Product Specification of: Medical Oxygen Sensor Type: M-04C

Part Number: 410206

Nuova ID: E-17/C



If the Sensor is dropped

If a sensor is dropped, please check for visible mechanical damage of the sensor or if the sensor is leaking electrolyte. If this is the case, do not try to install the sensor into the device. Take safety precautions and immediately dispose the cell.

If the sensor shows no visible damage it should be placed in quarantine for at least 24 hours. Afterwards a follow-up check made by a two point calibration should be done.

Installation in Device

A gas tight sealing is ensured when the sensor is screwed in hand-tight. Don't use any mechanical tools to install the sensor.

Using excessive force may damage the sensor.

Optimal mounting position of the sensor is when the gas sensing area faces downwards. A horizontal position is acceptable. It is not recommended to use the sensor with the gas sensing area facing upward.

Connection should be made via recommended electrical and mechanical connectors only. The specified load resistance must be taken into account. Soldering the sensor will damage it and void the warranty. Please contact ITG for further information.

Since temperature has an influence on the output signal it is not advisable to place any heat or vibration sources (i.e. electric- pumps or valves, coolers, etc.) in close proximity to the cell.

Avoid proximity of the cell to any EMC radiating units as those emit frequencies which might interfere with the sensor's electrical connection and connecting cable.

RFI/EMI Susceptibility

ITG sensors contain metal parts and might be susceptible to RFI or EMI. Before use in MRI environments please contact ITG for further information.

Disposal

At the end of the sensors lifetime the sensor should not be disposed of in normal public waste as it may contain hazardous materials and caustic electrolyte (for more information refer to the PSDS). Please contact your local authorities for environmental legislation to relevant local waste disposal.

This data sheet is subject to change without prior notice

Specification_M-04C_Rev_1

Page 4 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 .: Fax: +49 (0)3841 220052 2 .: E-Mail: info@itg-wismar.de www.itg-wismar.de



Locking latch

Product Specification of: **Medical Oxygen Sensor**

Type: M-04C

Part Number: 410206

Nuova ID: E-17/C




TECHNICAL DRWAING

Sensor	ITG Part number	Color			Electrical pin assignment		
Selisoi		Сар	Housing	Threaded ring	Pin 1	Pin 2	Pin 3
M-04C	410206	white	white	white	minus (-)	minus (-)	plus (+)


Top view

Bottom view

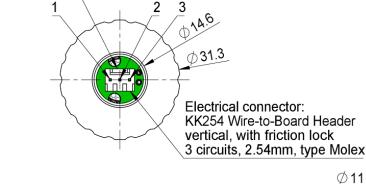

gas inlet diameter

Front view

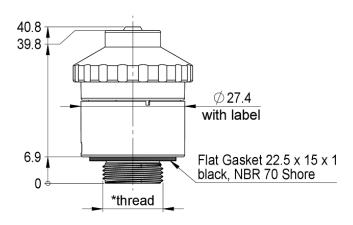

[* thread is compatible with DIN 13 M16x1 and UNEF 5/8-24]

Dimension unit: mm Dimension tolerances: linear $\pm 0.5~\text{mm}$, diameter $\pm 0.3~\text{mm}$

This data sheet is subject to change without prior notice

Specification_M-04C_Rev_1

Page 5 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 .: Fax: +49 (0)3841 220052 2 .: E-Mail: info@itg-wismar.de www.itg-wismar.de